



REGULAR SIZE MODEL

User/Caregiver Manual

INTRODUCTION

Thank you for buying the FOLD & GO Wheelchair, which is designed and manufactured by OneKubedDESIGNS, which is Trademarked. Before operating this electric wheelchair for the first time, please read this manual carefully. This instruction manual will help you to understand the performance, operation, features, and routine maintenance. Please refer to this manual and or our website for how to videos on operating your new chair.

Wheelchair Easy Maintenance

There are a few things to do to keep your FOLD & GO WHEELCHAIR running in top condition. Included with your chair is a mini travel toolkit. We suggest you keep this tool kit in the storage bag under the seat, that way it is always with you. Every 3 months we recommend you examine the bolts and screws of your chair to make sure everything is tight and locked in place. If you take your chair on the airplane with you, then make sure to examine the bolts and frame for ANY damage that might have happened from baggage handlers and the flight.

Your wheelchair will work best when you take care of it, just like you would your automobile. On your car, you keep oil in the engine, you keep air in the tires, you fill it with gas to keep it running, and you keep it clean so it will not rust. Your FOLD & GO WHEELCHAIR needs a similar type of regular easy maintenance.

User Safety

Please don't use or operate this electric wheelchair until you have read and understand these instructions. If you can't understand these instructions, please contact us for technical guidance and help. Otherwise it may cause harm or danger to yourself by not operating the chair correctly.

Always turn off the Joystick when entering and exiting the chair. Practice driving operations with the help of family, friends or a caregiver, until you are independent and skilled in performing these operations while you are exploring the world we live in.

Caregiver Safety

Each FOLD & GO Wheelchair can be operated in manual or power mode. If you wish to manually push your client make sure to move the red levers on each rear wheel motor to disengage the electromagnetic brakes. Also make sure that the top headrest frame is locked into place, and not in the downward storage position. Before each movement tell your client what you are doing to avoid any risk of accidental injury because they are not prepared to be moved. Cooperate with your client and their physical limitations.

PLEASE READ BEFORE OPERATING YOUR WHEELCHAIR



DO NOT OPERATE OR DRIVE THE WHEELCHAIR THROUGH WATERWAYS, INTO LAKES NOR OCEANS. DO NOT USE THE BATTERY IN A DIFFERENT CAPACITY OTHER THAN WITH YOUR FOLD & GO WHEELCHAIR. KEEP TOOLS AND OTHER METAL OBJECTS AWAY FROM THE POSITIVE AND NEGATIVE ENDS OF THE BATTERY.

MODEL COMPONENTS

Please take a moment to make sure you have all of the following parts with your FOLD-N-GO Wheelchair. If you are missing ANY components please contact us immediately at support@foldandgowheelchairs.com. Accessories are sold separately.



WHEELCHAIR

FOLD-N-GO Wheelchair in your desired color



SEAT CUSHIONS

Two cushions, one for your seat and one for the backrest



BATTERY

One battery to be placed in the side arm of the chair, a fake plastic housing for the empty slot



CHARGER

Power charger with connection to joystick, and power cord to plug into the wall outlet



JOYSTICK CONTROLLER

One joystick to control movement of the chair. Can be placed on the either the right or left armrest



SELT BELT

Portable strap for safety while riding in your chair, and to wrap up the armrests when folded



PARTS OF YOUR FOLD-N-GO WHEELCHAIR



INSTRUCTIONS

How to Install & Operate



1. Battery Installation

Slide the skinny part of the battery into the slot on the side arm. Make sure the battery is inserted all the way, so the red and black plug connectors show at the bottom of the side arm case. Then plug in the cable from the black box under the seat into the battery. Match the red with red and black with black part of the plugs.



2. Joystick Installation

The Joystick can be installed on the left or right armrest. Insert the Joystick into the tube of the armrest. Tighten the Screw bolt on the bottomside of the armrest. Make sure this bolt is snug to hold the Joystick in place.



3. Manual Mode - No Brakes

Move the red lever on the inside of each rear wheel motor, push it forward until you feel a click. Then the wheel will spin free, so the chair can be push just like a manual chair. Repeat the same process for both wheels.



4. Power Mode - Electromagnetic

Move the red lever on the inside of each rear wheel motor, pull it backward until you feel a click. Then the wheel motor will be engaged and power will move each wheel. Repeat the same process for both wheels.

How to FOLD the FOLD-N-GO Wheelchair



- Turn off the Joystick then remove it and put it in the storage bag under the seat for safe keeping.
- Rotate both armrests up and over towards the back of the chair.
- Pull steel wire under the backend of the seat cushion.
- Push the backrest forward toward the seat cushion.
- Place one hand on the top of the backrest, and with your other hand pull the footrest up.
- If you need additional storage room, you can push the levers on the top of the backrest and fold down.

How to UNFOLD the FOLD-N-GO Wheelchair



- Push the levers on the top of the backrest to unfold and rise it back into place.
- Place one hand on the top of the backrest, and with your other hand push the footrest down.
- Pull steel wire under the backend of the seat cushion.
- Push the backrest back toward the seat cushion, until it locks in place.
- Rotate both armrests up and over towards the front of the chair.

Specifications of the Regular FOLD-N-GO Wheelchair

Net Weight	46 LBS
Weight Capacity	265 LBS
Motor	180 Watt x2 Brushless
Battery	Lithium-Ion 24V/10AH Standard
Max Speed	4 MPH
Turning Radius	31.5 inches
Grade/Slope	12 degrees
Charging Time	5 Hours
Driving Range	12 Miles per Battery
Overall Length	34 inches
Unfolded Width	24 inches
Folded Width	10 inches
Seat Width	16 inches
Back Rest Height	14 inches
Seat Height	18.5 inches
Brake Type	Electromagnetic



Joystick Functions



Movement	Direction to Move Joystick
FORWARD	Move the joystick FORWARD
BACKWARD	Move the joystick BACKWARD
RIGHT	Move the joystick RIGHT
LEFT	Move the joystick LEFT
STOP	Release your hand from the joystick

Joystick Diagnostics

When the LED light on the Joystick flickers, it means that there is something wrong with the wheelchair. An alarm may also sound from the rear motor wheels, brakes, batteries, wire connections, or from the wheelchair itself. Below are the diagnostic codes to help you identify any problems you may be experiencing while using your FOLD-N-GO Wheelchair:

Flash or Beep	Description & Troubleshoot
Flash Once Slowly	Left rear motor wheel is not connected
Flash Twice Slowly	Right rear motor wheel is not connected
Flash 3 Times Slowly	Battery is not charging, the cables to the battery may be loose
Flash 4 Times Slowly	Check left rear motor, power cable is loose or not connected
Flash 5 Times Slowly	Check right rear motor, power cable is loose or not connected
Flash 6 Times Slowly	Check left brake motor, power cable is loose or not connected
Flash 7 Times Slowly	Check right brake motor, power cable is loose or not connected
Flash 8 Times Slowly	Contact FOLD-N-GO Customer Service
Flash 9 Times Slowly	Joystick cable is loose to black junction box under the seat
One Short Beep	You are trying to move the chair with the Joystick before it is ready
Beep in Any Direction	The rear wheels are in manual mode. Move the red lever backwards
Flash 4 times Quickly	Left motor is over heated, disconnect cable for 30 seconds then reconnect
Flash 5 times Quickly	Right motor is over heated, disconnect cable for 30 seconds then reconnect

DRIVING CONDITIONS & ENVIRONMENT

- Don't try to drive the FOLD-N-GO Wheelchair when you are on a transport vehicle, like a bus, subway, trains, planes, etc.
- While using public transportation vehicles, look for a place to fix the wheels to the floor of the bus. This will help you avoid harm that can be caused by the vehicle's sudden stop or a traffic accident.
- In many countries and regions, power wheelchairs are not allowed on public roads. The user must realize that driving a wheelchair on the road instead of the sidewalk is dangerous.
- For easy recognition at night, a reflective article can be placed on the FOLD-N-GO Wheelchair. The user can also wear reflective clothing.
- When confronting with other vehicles, please ensure that the driver has noticed you. Before you continue to operate your wheelchair, make eye contact with the driver, and let the driver know your intention until you are sure that it is safe.
- When driving the FOLD-N-GO Wheelchair keep the chair in balance, in order to avoid being bent over in the forward or backward position.
- If you desire to modify or adjust the FOLD-N-GO Wheelchair, please consult a reputable dealer in advance. The modified wheelchair may need to have additional adjustment to keep the chair working properly.
- Making any modifications to your FOLD-N-GO Wheelchair will void ALL warranties.
- When you are entering or exiting the FOLD-N-GO Wheelchair, you must turn off the power supply. Otherwise, if you touch the joystick, it may lead to the unexpected movement.
- The FOLD-N-GO Wheelchair is not suitable for staircases, or on an escalator. Pay attention when using the chair on the elevator.
- When an elevator is not available, fold up the chair to move it from floor to floor by way of the stairs.



212 E Meadowlark Street, Austin, TX 78734

Customer Service: 512-817-FOLD

support@foldandgowheelchairs.com

www.foldandgowheelchairs.com